



**A-dec Headquarters**  
 2601 Crestview Drive Newberg, Oregon 97132 USA

Tel: 1.800.547.1883 Within USA /Canada  
 Fax: 1.503.538.0276 Within USA /Canada

Tel: 1.503.538.7478 Outside USA /Canada  
 Fax: 1.503.538.5911 Outside USA /Canada  
[www.a-dec.com](http://www.a-dec.com) / [www.a-dec.biz](http://www.a-dec.biz)

**A-dec 311 (Version B)  
 Dental Chair  
 Mini Template**

86.0551.00 Rev A 2015-04

**A-dec 311 Plumbing Elevations:**

5/8" (16 mm) O.D. tubing for wet systems.  
 40 mm O.D. tubing for dry/semi-dry systems  
 Central vacuum only. (see General Utility Notes)

1-1/2" (38 mm) nominal pipe for gravity drain (when required)

1" (25.4 mm)

2" (50.8 mm)

1/2" (13 mm) nominal pipe compression fitting with right angle manual shut-off valves for air & water.

1/2" (13 mm) conduit for electrical. (supplied by contractor; check amperage requirements)

1" (25.4 mm)

5" (127 mm) max. height

Floor Surface

Floor Surface

PLEASE NOTE: This drawing is for reference purposes ONLY. It is NOT TO SCALE.

**A-dec Track Light**

7-1/2" (191 mm)

77-1/2" (1969 mm)

58" (1473 mm)

27-3/4" - 31-1/2" (705 mm - 800 mm)

15"-39" (381 mm-991 mm)

CAUTION: Please see note below.

7-1/2" (191 mm) from track light to centerline of mounting post. Positioning ensures adequate space for both mounting plate and mounting plate cover.

25" (635 mm)

A-dec 577L Light

A-dec 6300 Light

NOTE: If the track opposite the transformer is closer than 9-1/2" (241 mm) from a wall, the trolley and post assembly must be installed before the track is installed on the ceiling.